
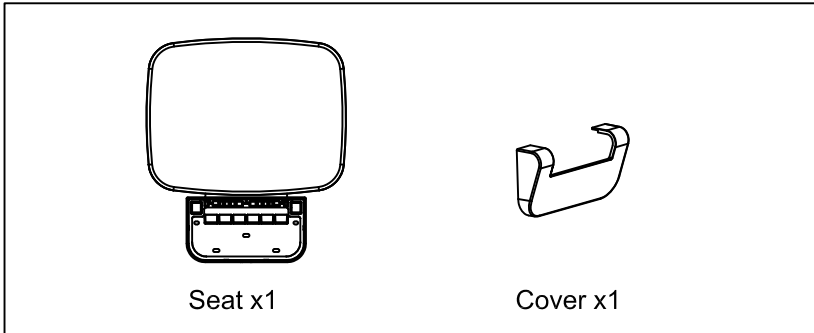


# Shower Seat Chrome & White Installation Instructions 41200100830

**PARTS LIST:**

**TOOLS REQUIRED:**


- **Maximum load: 160kgs**
- **The strength of this seat relies on correct installation. It is important that the seat is attached to the wall using appropriate fixings for the type of wall, and that the wall is strong enough. Please take care when selecting the fixing method.**
- **Care should be taken to ensure all five screws (not supplied) are firmly fixed to evenly distribute the load. Not doing so could result in undue stress on the product with the possibility of failure and risk to the end user.**
- **When first using the seat take care when sitting down to ensure that the wall fixings are secure. Wall fixings which aren't strong enough or a fixing method with insufficient strength can cause the seat to come away from the wall and lead to the user falling.**
- **Use warm soapy water or mild detergent for cleaning the chromed ABS cover, Do not use strong chemicals or abrasives.**

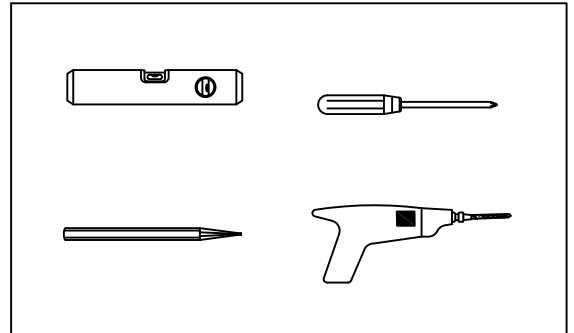